



**TP 05**

**VENOLYMPHATIC DRAINAGE SYSTEM**



***TECHNICAL SPECIFICATIONS***

<b>Weight:</b>	6.5 kg	<b>Power :</b>	44 W	<b>Sound Level:</b>	49 dB<SL<52 dB
<b>Dimensions :</b>	11*40*31 cm	<b>Voltage / Frequency:</b>	220 V/50 Hz		
<b>Compression range:</b>	15-150 mmHg			<b>ICE 6001-1# 94,2 00547, 02</b>	

***Pressure regulation mode:***

↳ Automated Double Gradient Compression (A.D.G.C. – International Patent)

***Durations Adjustments :***

↳ Inflation – Deflation – Rest and treatment time

***Accessories Inflation :***

↳ Two 5-connection levels enable inflation of 2 accessories simultaneously.

***Accessories components:***

Outer fabric: Nylon / Inner fabric: Polyurethane  
Fastening: welded parts

***Connecting:***

Elements / tubes: permanent to avoid leaks  
Pump / tubes: numbered connectors

***Accessories:***

- 5-chamber sleeves
- 5-chamber boots
- 5-chamber waist boots
- 5-chamber abdominal strap



***Comfort in use:***

Very soundproof TP 05 thanks to innovative design

**A UNIQUE 11-STAGE CYCLE**

***The TP05 accessories have 5 chambers and numbered connections from 1 (distal) to 5 (proximal).***

***The Automated Double Gradient Compression between the 5 chambers is observed with the pressure in the fifth chamber that is the one selected by the operator at the beginning of the cycle.***

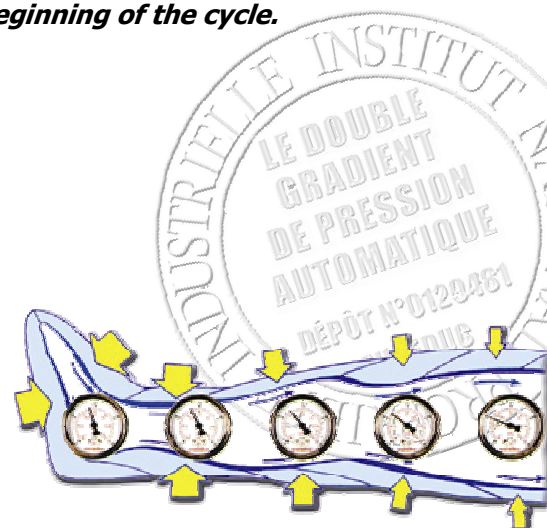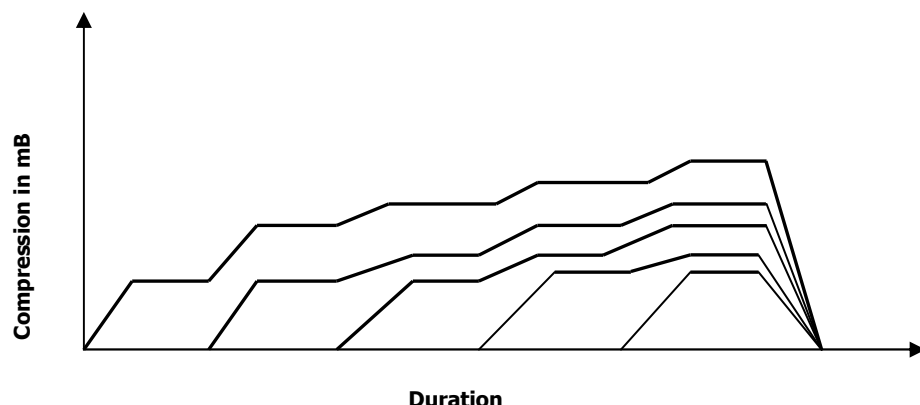

- Stage 1** ⇒ Chamber #1 inflation  
**Stage 2** ⇒ Working time. The chamber #1 will keep inflated until the end of stage 11  
**Stage 3** ⇒ Chamber #2 inflation + additional pressure in chamber #1  
**Stage 4** ⇒ Working time. The chamber #2 will keep inflated until the end of stage 11  
**Stage 5** ⇒ Chamber #3 inflation + additional pressure in chamber #1 and #2  
**Stage 6** ⇒ Working time. The chamber #3 will keep inflated until the end of stage 11  
**Stage 7** ⇒ Chamber #4 inflation + additional pressure in chamber #1, #2 and #3  
**Stage 8** ⇒ Working time. The chamber #4 will keep inflated until the end of stage 11  
**Stage 9** ⇒ Chamber #5 inflation + additional pressure in chamber #1, #2, #3 and #4  
**Stage 10** ⇒ Working time. The chamber #5 will keep inflated until the end of that stage  
**Stage 11** ⇒ Simultaneous deflation of all chambers – Resting time

## MAIN INDICATIONS

- Limbs edemas: Lymphedema - Phlebedema
- Thrombosis prevention – phlebitis after-effects
- Circulatory problems
- Heavy legs
- Post-op regain
- Body profiling

## MAIN CONTRA-INDICATIONS

- Acute venous thrombosis
- Inflammatory / infectious estate
- Cardiac / renal insufficiency

## ACCESSORIES

TP05	Lymphatic drainage system	
M5PPM	Sleeve (small)	Ø 50 cm * 71 cm
M5PGM	Sleeve (large)	Ø 58 cm * 77 cm
M5PSM	Sleeve (size on demand)	Ø X cm * X cm
B5PPM	Boot (small)	Ø 70 cm * 85 cm
B5PGM	Boot (large)	Ø 80 cm * 92 cm
B5PSM	Boot (size on demand)	Ø X cm * X cm
H5P	Waist boots	Ø 80 cm * 106 cm
H5PSM	Waist boot (size on demand)	Ø X cm * X cm
SPK	Abdominal strap	L: 56 cm * l: 37 cm
SPKSM	Abdominal strap (size on demand)	L: X cm * l: X cm
GHB	Hygiene sheathes (100u batch)	



DOCTP05GB2011-02-04